CANINE PARVOVIRUS ANTIGEN TEST KIT

II. TEST PRINCIPLES

WITNESS™ Parvo detects all strains of canine parvovirus (CPV) shed in the feces. Positive results with WITNESS™ Parvo indicate the presence of canine parvovirus.

The WITNESS™ Parvo is a simple test based on Rapid Immuno Migration (RIM™) technology. Canine parvovirus antigen is detected in canine feces. Sensitized particles will form complexes with the CPV antigen present in the sample (fecal extract). The formed complexes migrate along a membrane. The complexes are then captured on a sensitized reaction line where their accumulation causes the formation of a clearly visible pink/ red band. A pink/red band in the control window ensures that the test was performed correctly.

III. SAMPLE MATERIAL Canine fecal material is required for this test.

IV. SAMPLE STORAGE

Stool samples should be tested not longer than 24 hours after collection, if stored at room temperature. If testing is further delayed, samples should be kept refrigerated (up to 7 days at +2 °C to 7 °C; 36 °F to 46 °F).

For long term storage, samples should be kept frozen (-20 °C; -4 °F or colder). V. KIT CONTENTS

- 5 Pouches, each containing 1 test device and desiccant. 5 Disnosable extraction devices
- 1 Extraction Buffer bottle (12.5 mL).
- Instructions for use.

VI. PRECAUTIONS

- Do not use this kit or any of its components after the expiration date.
- Kit should be stored at +2 °C 25 °C (+35 °F 77 °F). Kit should not be frozen. Use the test immediately after the pouch is first opened (within 10 minutes).
- Avoid touching or damaging the membrane in the sample wells or the results windows.
- The WITNESS™ device should be placed on a flat, horizontal surface while performing the test.
- Use a separate extraction device for each sample.
- Hold extraction device and buffer bottle vertically when dispensing sample and buffer respectively.
- Handle all samples as biohazardous material.
- Extraction buffer is preserved with sodium azide
- For veterinary use only.
- VII. TEST PROCEDURE

Important: Allow sample drops to fall onto the membrane at window (1). Do not touch the device tip or

the sample drops directly to the membrane.

1. SAMPLE COLLECTION Remove the top portion containing the cotton swab, from a sample

- extraction device. Dispense Extraction buffer into the base of the sample extraction device
- until the buffer reaches the graduation mark. Taking the top portion of the sample extraction device, coat the cotton swab
- with a thin layer of feces from a stool sample, or a rectal swab Insert the swab into the dispensed extraction buffer in the base of the sample
- extraction device. Securely fit the top portion onto the base and tap to mix for 5 seconds to ensure good sample extraction.
- 2. SAMPLE APPLICATION Tear open a pouch provided and place the test device on a flat horizontal surface.



- Snap off the top of the sample extraction device (at the blue line) containing the extracted fecal sample.
- Invert and hold the sample extraction device in a vertical (upright) position and squeeze the base gently to dispense five drops of sample to the test well. Leave the test device flat during migration of sample/reagent complex




- through the reading windows.
- 3. READING TEST

After 10 minutes, observe the presence or absence of pink/red bands in

the reading windows (2) and (3). Note:




It is possible to read the test before 10 minutes if two pink/red bands are clearly visible in the results windows (2) and (3).

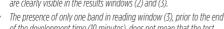




of the development time (10 minutes), does not mean that the test

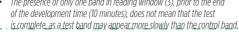



Valid Results Test is valid if a pink/red band is present in reading window (3).

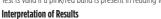



- Positive: One band in reading window (2), with one band in
- window (3): sample is positive for CPV antigen. Negative: No band in reading window (2), with one band in
- window (3): sample is negative for CPV antigen.
- Invalid test: No band in control window (3).



- Note: Samples with low levels of CPV antigen may produce a lighter colored yet distinct pink/red band.
- A test result should always be interpreted in the context of all available clinical information and history for the dog being tested. If clinical signs warrant, it is recommended to retest using a fresh sample.

SYMBOL DESCRIPTIONS

LOT Batch Code

ECIREP Authorized Representative in the European Community Use by Date

Consult Instructions for Use



Serial Number IVD In Vitro Diagnostics

Temperature Limitation *** Manufacturer (storage temperature range)

i

zoetis 7netis Inc

Kalamazoo, MI 49007, USA U.S. Vet. License No. 190

www.zoetis.com EC REP

SYNBIOTICS EUROPE SAS, a wholly-owned subsidiary of Zoetis Gerland Plaza, BAT. E,

23 Rue Pierre Gilles de Gennes,

- 69007 Lyon, FRANCE For product information call:
- US VMIPS 1-888-963-8471 Canada 1-800-461-0917